


This tool can be used to update the firmware on the Zephyr ISM BioHarness devices that are part of the Zephyr Team System solution. The linked firmware files are the latest for left side or normal front facing strap orientation.

Step One:

Select the Firmware file (for side or front):

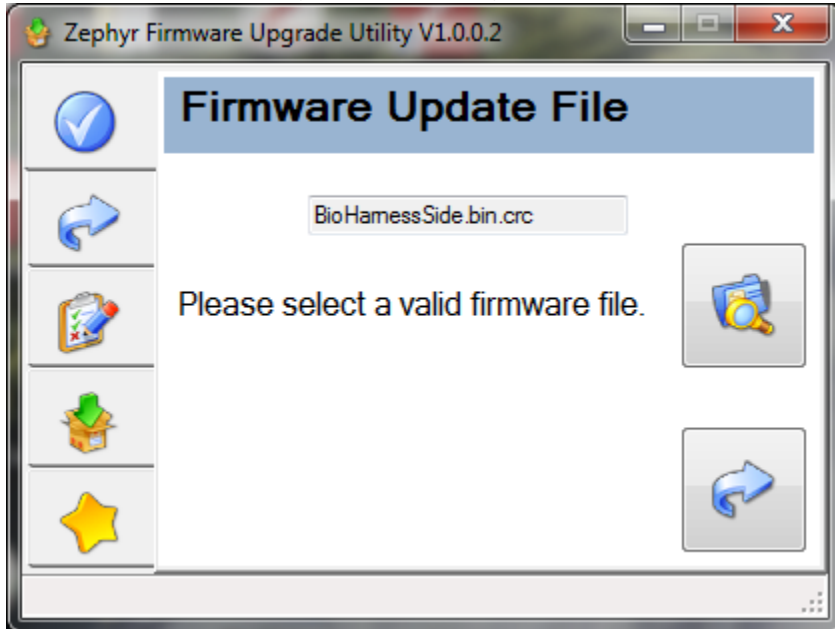

Then click the arrow.


Step Two:

Select the BioHarness that is attached to the computer.




Then click the arrow.


Step 3:
Click the program button in the bottom right.

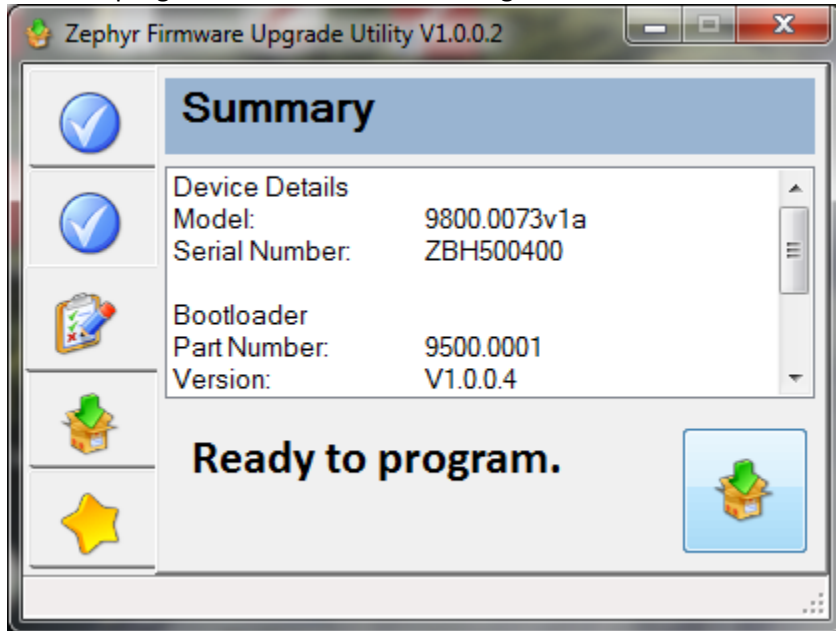

Step 4:
May take about 20 seconds to start programming.

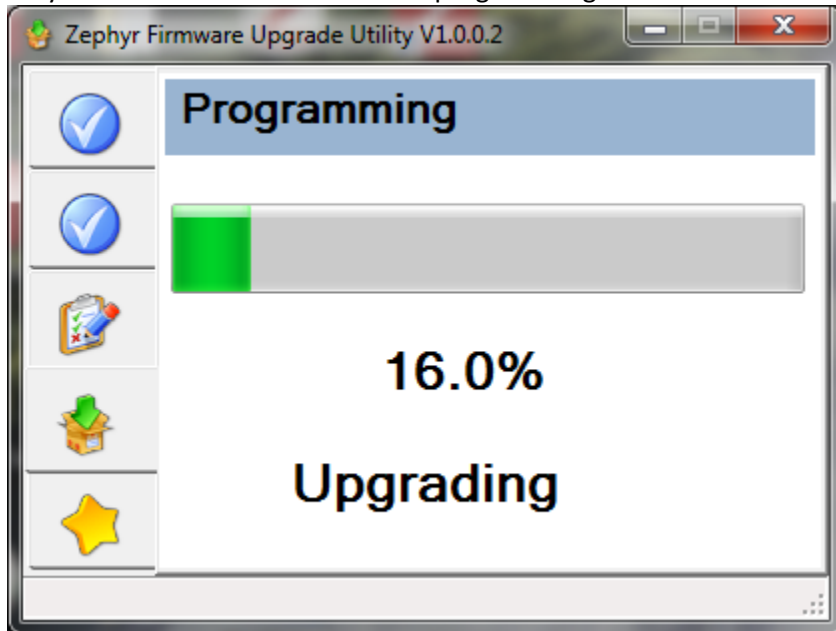

When finished:

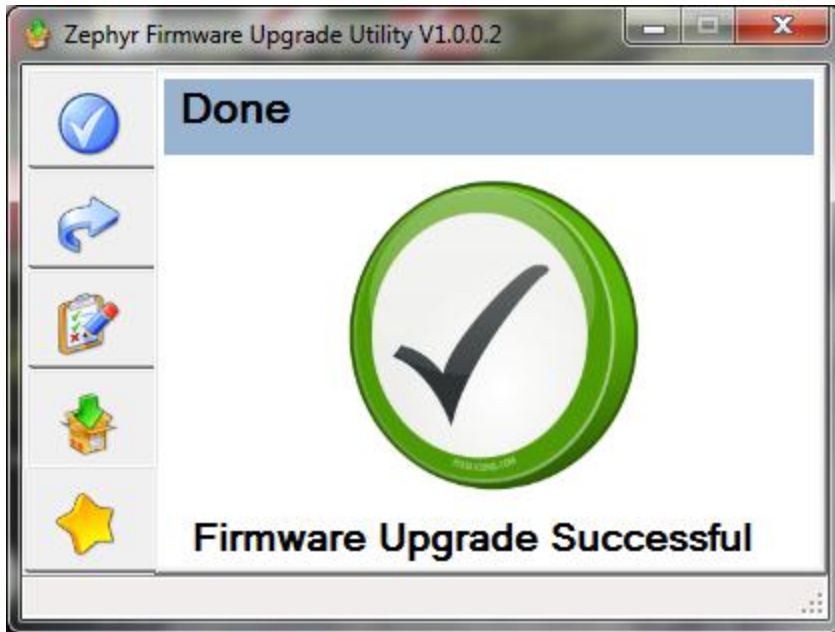

The BioHarness is now configured for the side strap.

You can also check the currently installed firmware version on any of the devices by scrolling down in step 3 above (before re-programming the device):

